

**PRODUCT INSTRUCTION SHEET**

REFERENCE: Oakham Chair ~ '700' (with winged back)  
'700NW' (without winged back)

**DESCRIPTION**

- Brown vinyl, metal frame, with or without winged head support and with lumber support.
- Generously padded and covered in vinyl for easy cleaning.
- Completely adjustable (overall height, seat depth and base height).

**TECHNICAL SPECIFICATION**

Overall height:	42.5" – 47.5" (1080mm – 1206mm)
Width between arms (outside):	22.5" (572mm)
Width of Back with Wings:	24"(635mm), w/o Wings: 19.5" (495mm)
Overall depth:	24" (610mm)
Seat height min-max:	16.5" – 21.5" (420mm – 550mm)
Seat depth:	17.5" – 20.5"
Seat Width:	19.5" (495mm)
Back height:	26" – 28.5" (660mm – 724mm)
Arm height:	25" – 30" (635mm – 762mm)



**USAGE REQUIREMENTS**

1. When adjusting the legs, ensure that there is no gap between horseshoe clip and frame.  
The clip spindle must be fully located right through the outside frame.
2. Check that the two wing nuts are fully tightened on the seat base and back support.
3. Do not exceed user weight of 165kg (26 stone).
4. Clean the frame with cream cleaner and upholstery with proprietary polish as required due to usage.