


# Opera® Classic and Classic Low Handset Guide

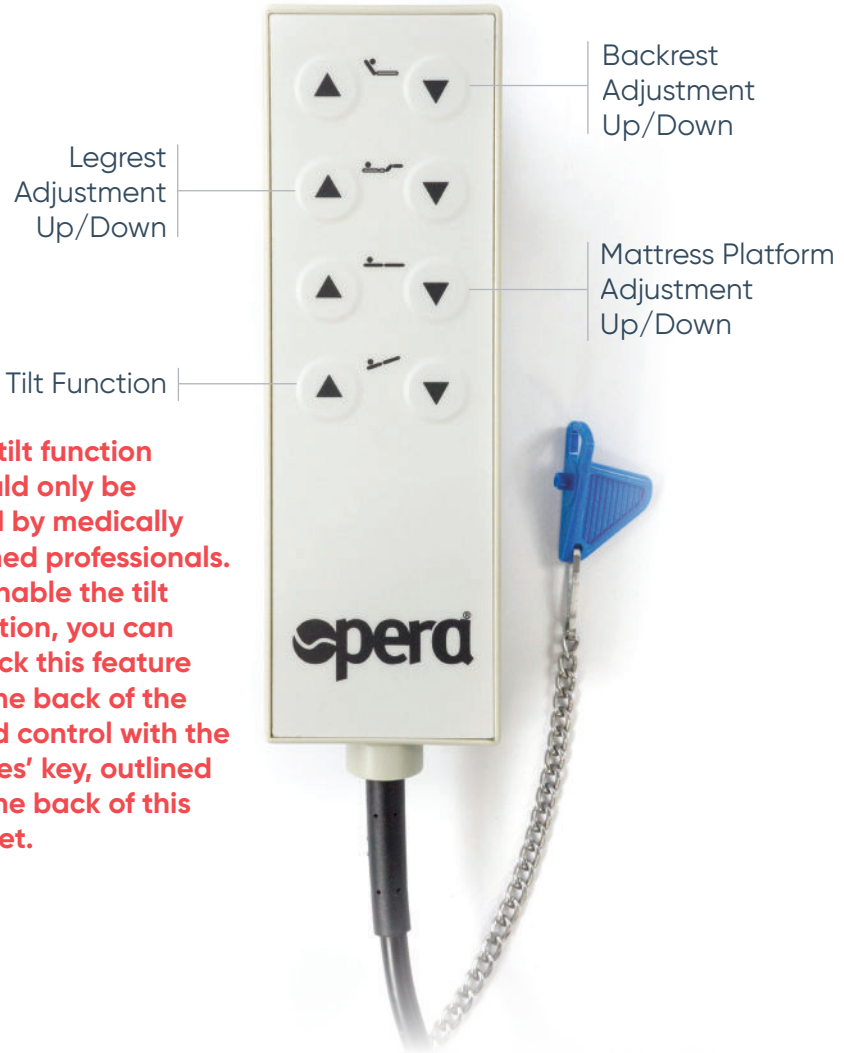

**The tilt function should only be used by medically trained professionals. To enable the tilt function, you can unlock this feature on the back of the hand control with the nurses' key, outlined on the back of this leaflet.**



## ATTENTION

To avoid damage, the handset should always be hung up (e.g. on the mattress base or side rail) when not in use.

Press only one button at a time, pressing multiple buttons at one time, may overload and damage the system.



All electric adjustment functions can be locked or unlocked by using the nurses' key (the blue key).



To lock the adjusting functions insert the nurses' key into the locking device and turn the key so that the locked padlock setting is selected. Turn the key to the unlocked padlock to unlock the setting.



The switching positions I and II are testing settings, used to check the safety of the bed during annual inspections, after repair work or each time the bed is put into storage.